


FAST FACE SHIELD

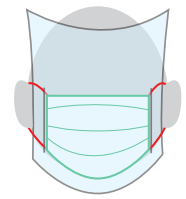

MATERIALS (for masks with vertical straps)

x1 mask

x1 sheet of clear plastic

- transparency film
- page protector
- cut 2-liter soda bottle

knife OR scissors



INSTRUCTIONS

1. CUT SLITS in plastic sheet.

Use your mask to measure the height of the slits you need to cut.

Cut one slit on each side, closer to the outer edges of the plastic. Note that the space between the slits is WIDER than the mask itself.


Be mindful of the placement of the cuts to make sure the coverage for eyes is in a good range.

2. FEED MASK LOOPS through holes


Mask is worn BEHIND the plastic.

Slide the slits up to the edge of the mask, plastic should CURVE outwards.

Plastic shield is meant to curve forward/away from the face for comfort and to lower fogging.

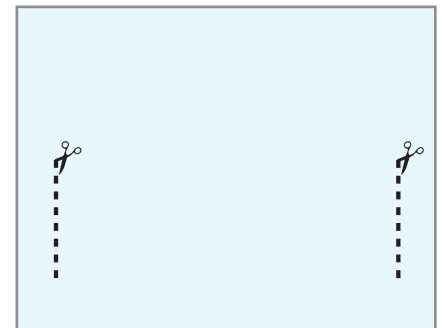

length of cut = height of mask

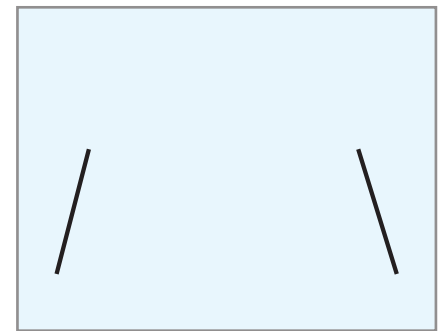

angling the slits may be helpful.

